

# GlucoPlus™ Management Software V1.0 Instruction

The GlucoPlus™ Management Software allows you to download your test results from your GlucoPlus™ Meter to your PC for review.

## 1 Minimum System Requirements

- A GlucoPlus™ Meter
- An IBM®-compatible computer with the following components:
  - Intel® Pentium® II 400 MHz processor or better
  - Minimum 64 MB of RAM or better
  - Minimum 60 MB of free hard disk space or more
  - Microsoft® Windows® 98/2000/XP
  - RS 232 port (or USB port with an adaptor)
- GlucoPlus™ Interface Cable
- Mouse and keyboard
- Printer for printing reports

## 2 Software Installation

2.1 Download the management software from our web site or load the GlucoPlus™ Management Software CD from your computer.

2.2 Choose the appropriate software version (Window® 98, 2000, or XP).


2.3 Click on SETUP.EXE to start.  
Press [OK] to begin the installation process (Fig.1).

Fig. 1




2.4 Click on the PC logo to install the management software.


Fig.2

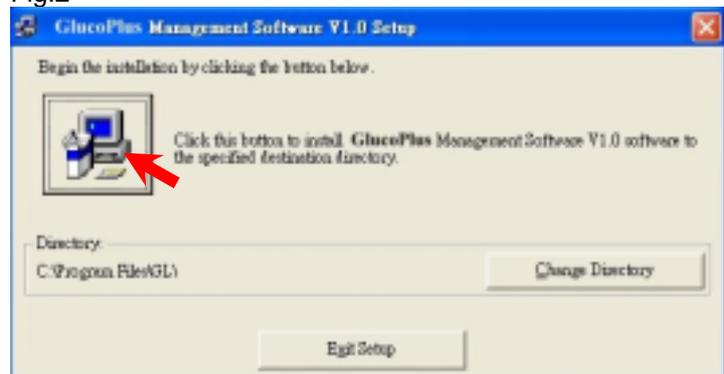


- 2.5 Select the GlucoPlus™ Management Software then press [Continue] (Fig. 3).

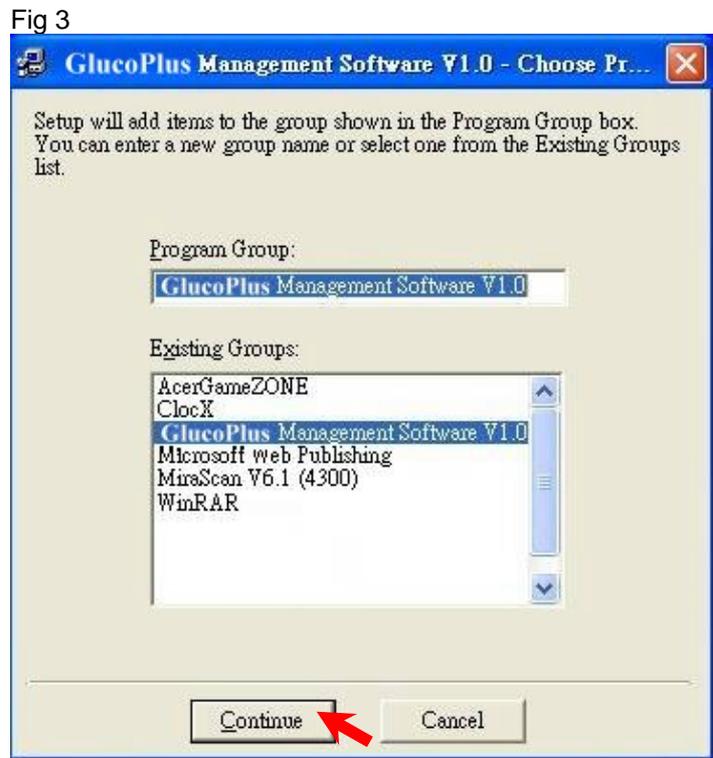

- 2.6 If a version conflict window is displayed (Fig. 4), click [No] to overwrite the earlier version of the management software.

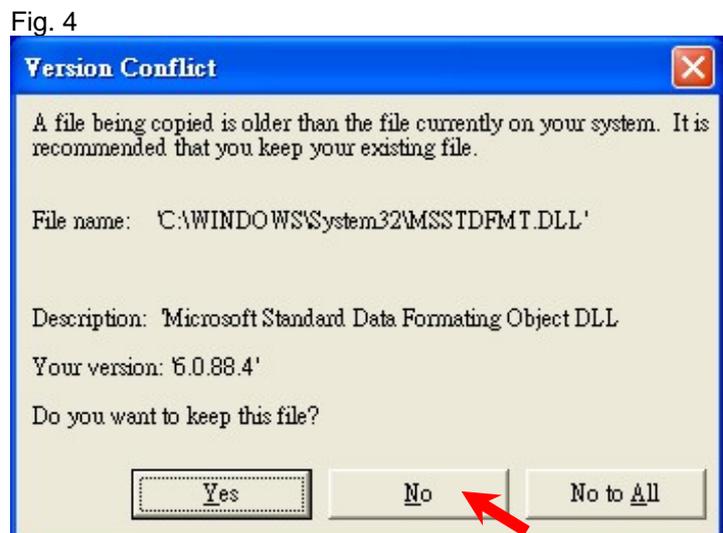

- 2.7 Click on [Cancel] to abort the installation process while installing the software.

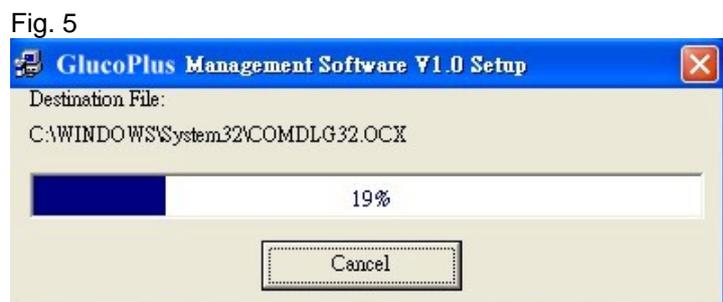

- 2.8 Press [OK] upon completion.



3.1 Find the directory where you have saved the GlucoPlus™ Management Software.

Fig. 7



3.2 This display appears on your screen (Fig. 7).

Enter your [User ID] and [Password], then press [Login] (Make sure to record and store your User ID and Password).

3.3 You may also use the management software by pressing [Login], but any data entered will not be saved.

Fig. 8

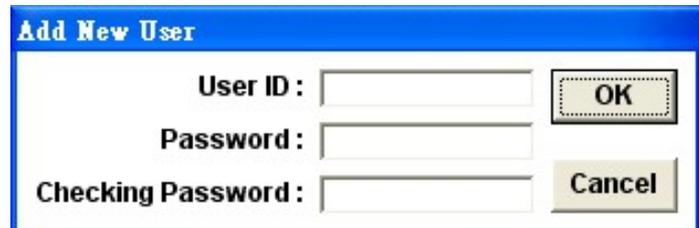


Fig. 9




3.4 After you have logged in successfully, the Welcome screen displays on the screen (Fig. 9)

3.5 Connect your GlucoPlus™ Meter to your PC with a Connecting Cable.

3.6

Click on [Start Readings] to download test results from your glucose meter onto your computer.

Fig. 10

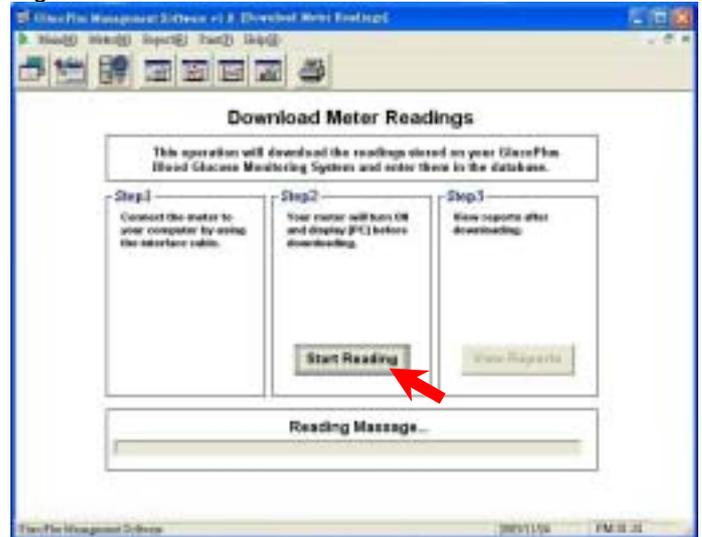

3.7

Click on [View Reports] to review your test reports after all the test results are transferred onto your computer.


Fig. 11



3.8

The Home window displays all test reports.

Fig. 12

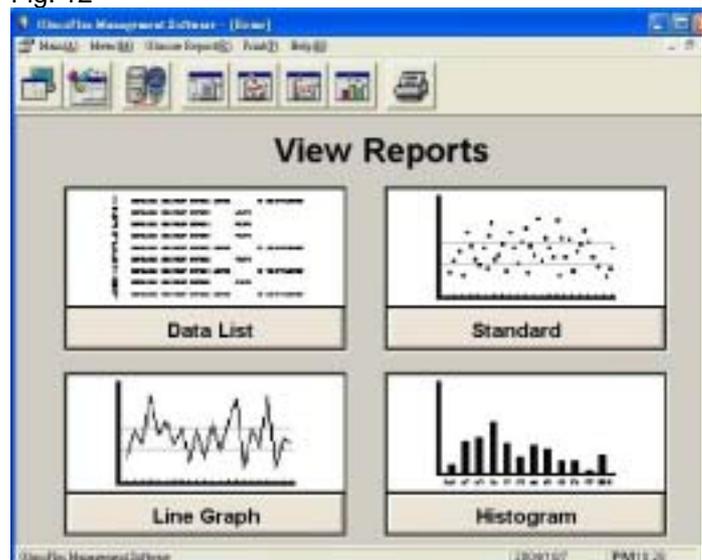


3.9

By reading the report, you must configure the transfer port, time/date format, unit of measurement, and the blood glucose limits through [Setup] under [Main] before using this program (Fig 13).

Fig. 13



- 3.10 Choose the proper port, data, glucose unit and glucose value limit before pressing save.
- Click on [SAVE] and then [OK] to confirm the settings. The window will return to the Home screen (Fig. 14)

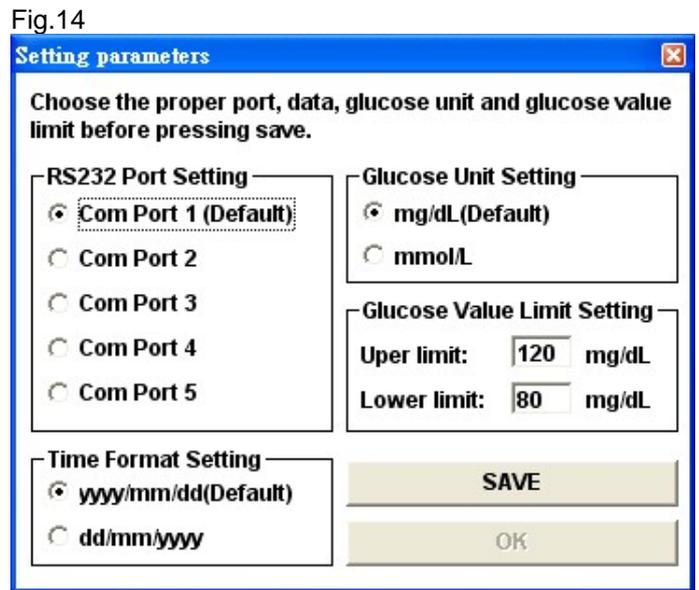


- 3.11 Click on the [Data List] icon to see details of your data(Fig. 15),

Fig. 15


Index	Glucose(mg/dL)	Month	Day	Hour	Minute
M0001	313	1	1	1	56
M0002	585	3	29	1	10
M0003	584	3	29	1	10
M0004	586	3	29	1	9
M0005	459	3	29	1	9
M0006	462	3	29	1	9
M0007	927	3	29	1	7
M0008	926	3	29	1	6
M0009	715	3	29	1	5
M0010	712	3	29	1	5
M0011	712	3	29	1	3
M0012	717	3	29	1	2
M0013	931	3	29	1	0
M0014	737	3	29	1	59
M0015	743	3	29	1	58
M0016	747	3	29	1	57
M0017	765	3	29	1	55
M0018	762	3	29	1	55
M0019	760	3	29	1	54
M0020	658	3	29	1	53

- 3.12 Click on the [Standard Day] icon to review your test results on a scatter graph.

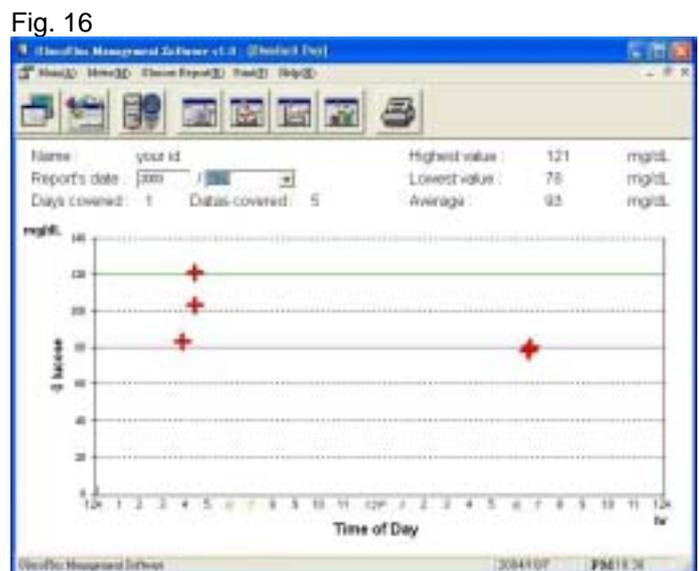


- 3.13 Click on the [Line Graph] icon to review your test results on a line graph.

Fig. 17

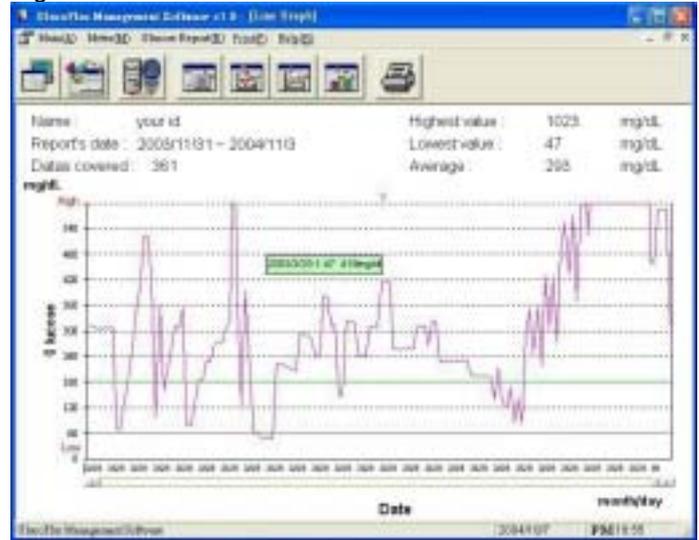


- 3.14 Click on the [Histogram] icon to review your test results on a histogram graph.

Fig. 18

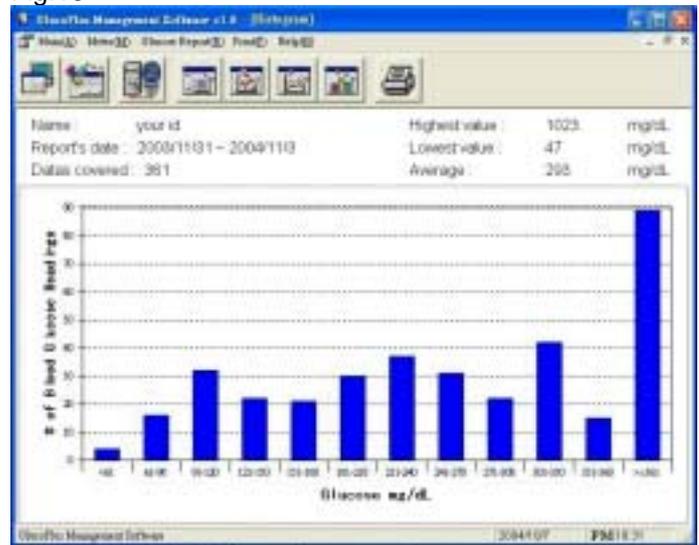

4. **Print your test report**  
Click on [Print] and then [Print Report] to print out your reports.

Fig. 19

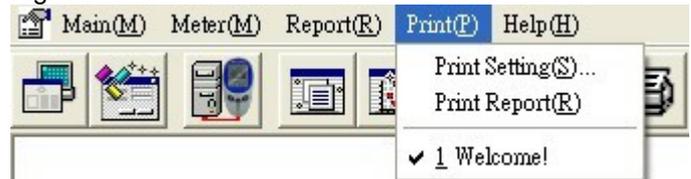

5. **Help**  
If you have any question on the software, please contact your local agent or click on [Our Web] under [Help] for our contact information.

Fig. 20

